

## Cleaning the Transducer

To disinfect the transducer it may be removed by means of the following procedure:

- 1 Rotating the turbine transducer anti-clockwise until the locating pip lines up with the small rectangular cut-out in the housing as shown below.
- 2 Gently pull the transducer away from the housing.
- 3 The transducer may now be immersed in warm soapy water for routine cleaning or immersed in cold disinfecting solutions e.g. PeraSafe for a maximum of 10 minutes (Alcohol and chloride solutions should be avoided). After cleaning, the transducer should be rinsed briefly in distilled water and dried.  
PeraSafe is available from Williams Medical Supplies. Product code: W10021.
- 4 Re-assemble the mouthpiece holder.

